
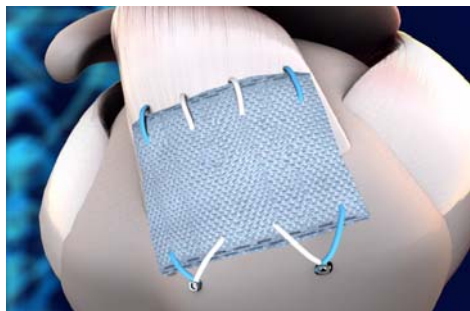
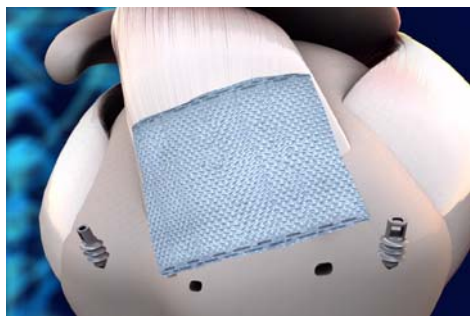


X-Repair Placement for Rotator Cuff Repair Augmentation

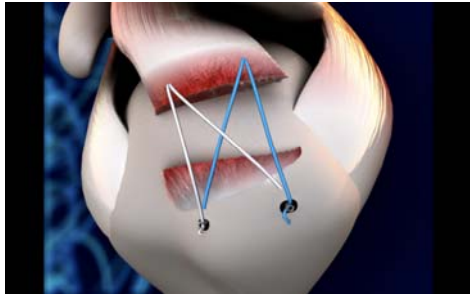

- Perform preparation of rotator cuff surgical repair including placement of sutures, but do not complete the repair.
- Select X-Repair size corresponding to the tear size and the length between the medial suture position and the lateral bone anchors. Orient X-Repair's LENGTH (welded end to welded end) along the medial-to-lateral direction of the tendon.
- Place sutures for attaching X-Repair to the tendon medial to the rotator cuff repair sutures.
- Complete rotator cuff repair.
- Use anchors dedicated to X-Repair lateral to the tendon repair, or suture anchors for rotator cuff repair with additional sutures.
- Attach X-Repair to the tendon using the medially placed sutures, and then laterally to bone via the suture anchors. Place sutures through the X-Repair internal to the ends of the device (>5 mm bite size).
- Impose appropriate tension on X-Repair to establish load sharing.